

# *Anatomage*

# TABLE

## **Cadavers**

How to Use the Pen Tool

### **Table of Contents**

How to Use the Pen Tool

1

## How to Use the Pen Tool

Step 1: Tap the Pen Tool icon



Step 2: Tap a default or custom pen color icon to select the desired color.

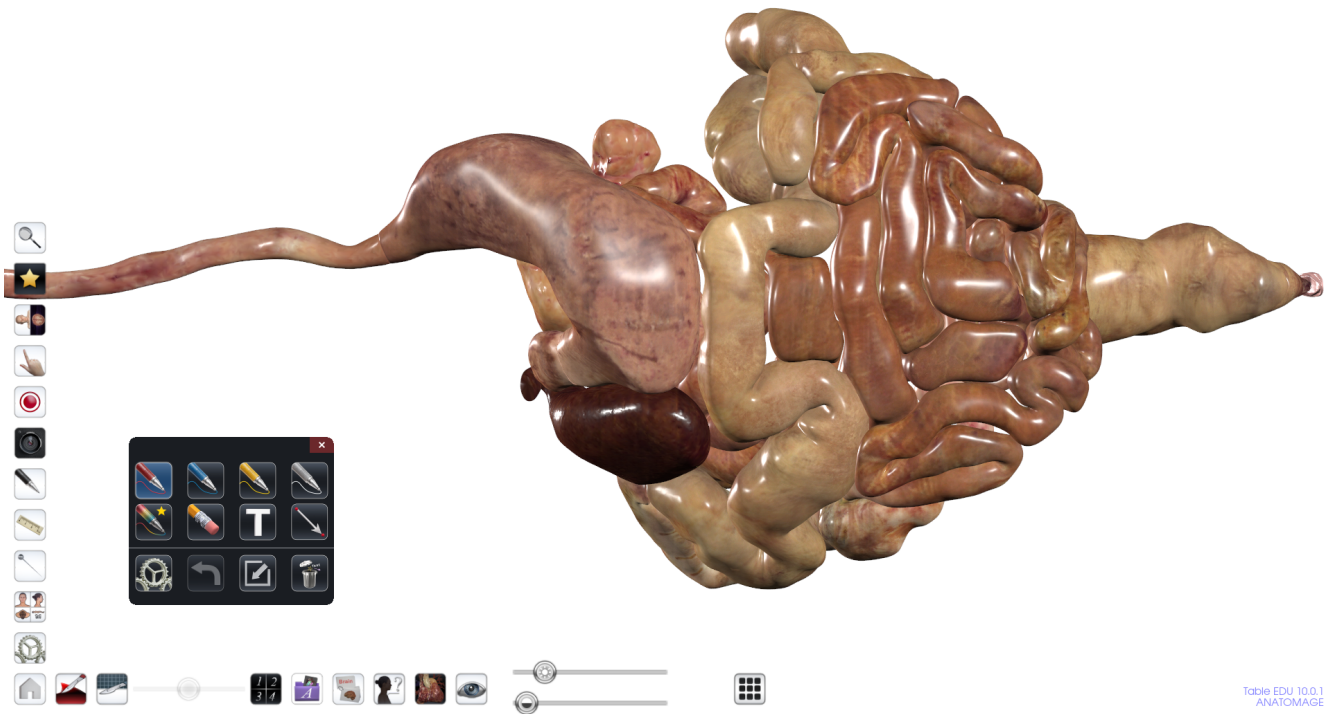


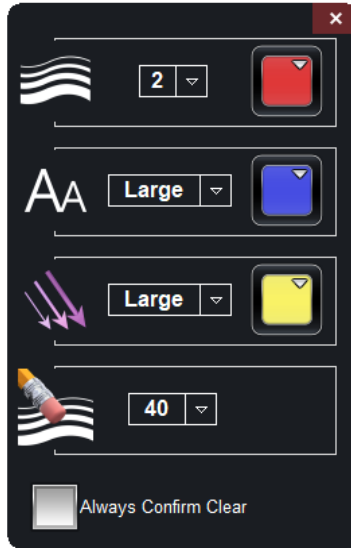


Table 10 - How to Use the Pen Tool

Step 3: To adjust a custom pen color, tap the settings icon  and select from the color

dropdown menu . You can also adjust pen stroke size, text size, arrow size, and eraser stroke size from this Pen Settings menu.



Step 4: Press and drag using finger to draw on the Table.

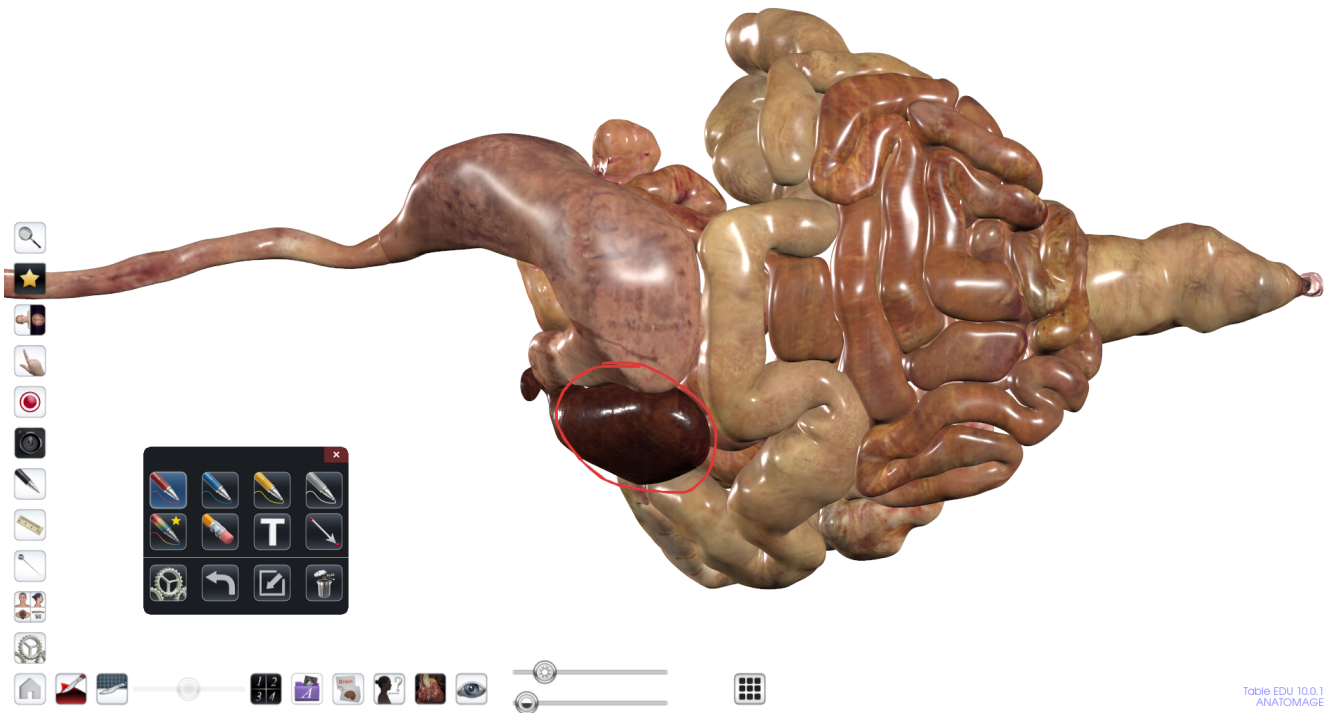

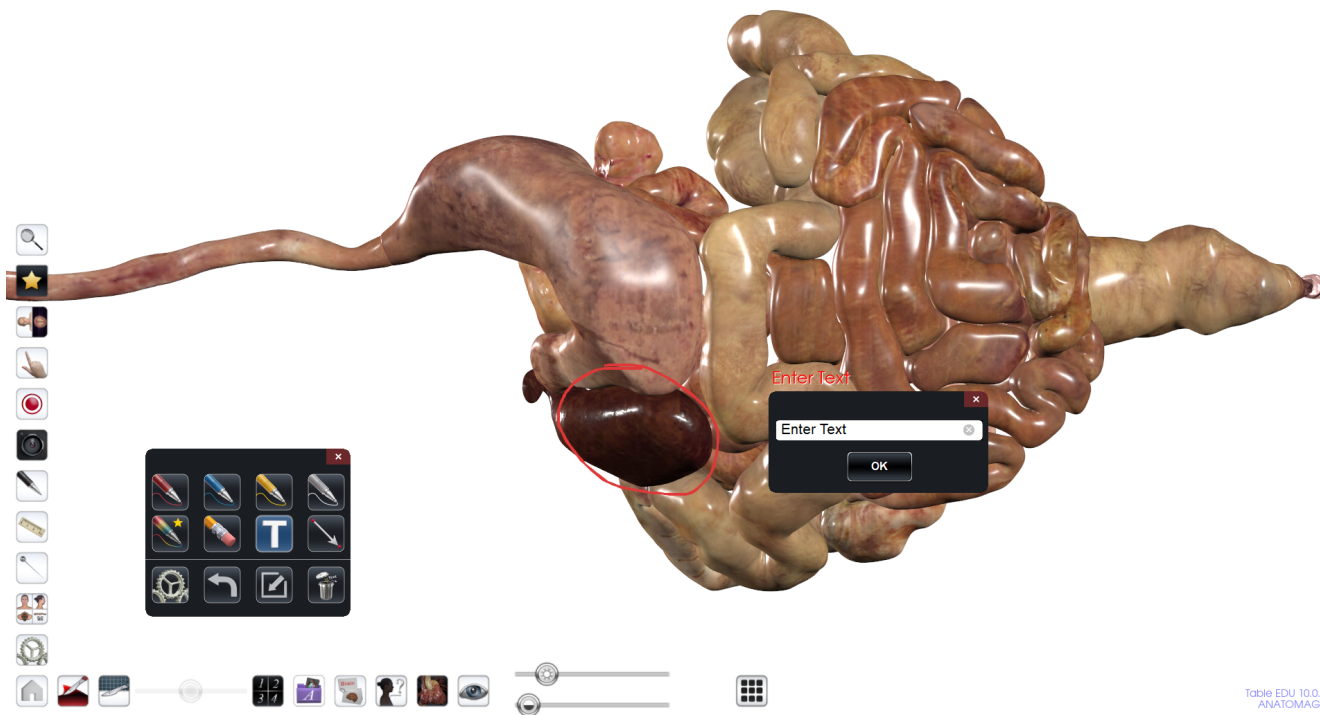


Table EDU 10.0.1  
ANATOMAGE

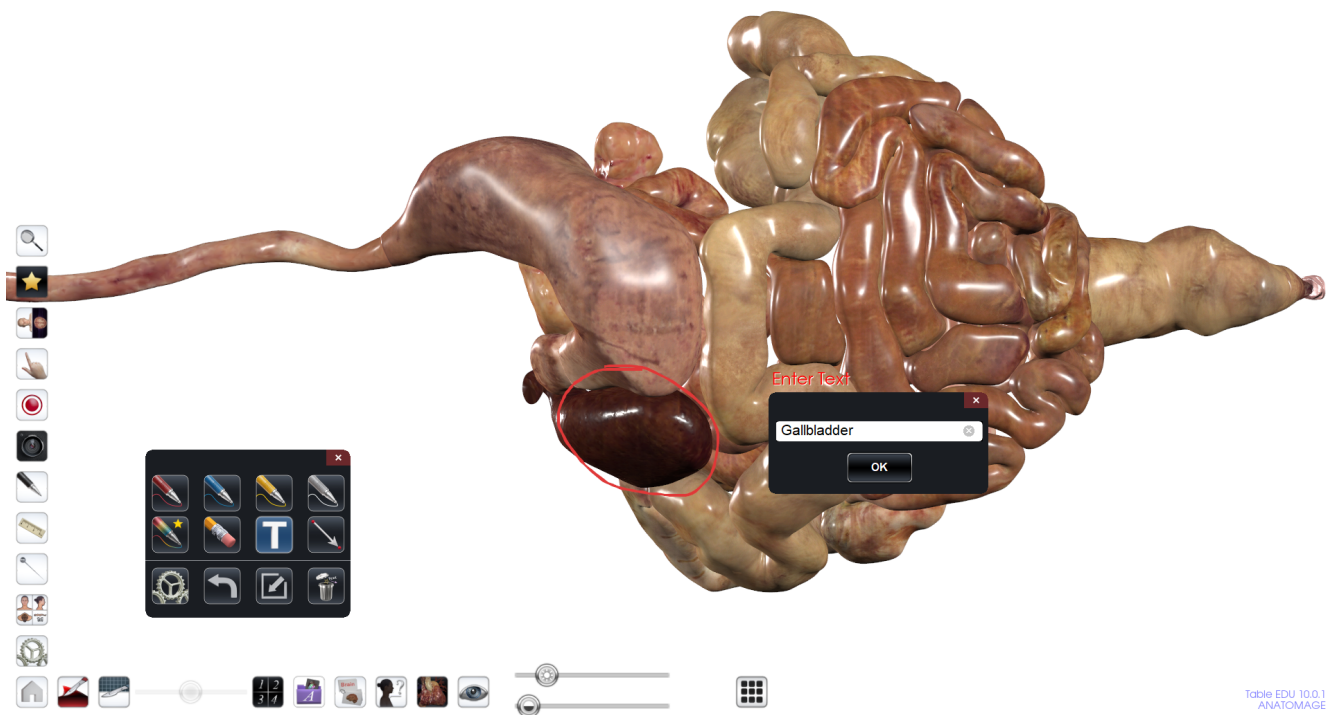
Step 5: Tap the text icon  to place text on the Table, then tap on the Table.




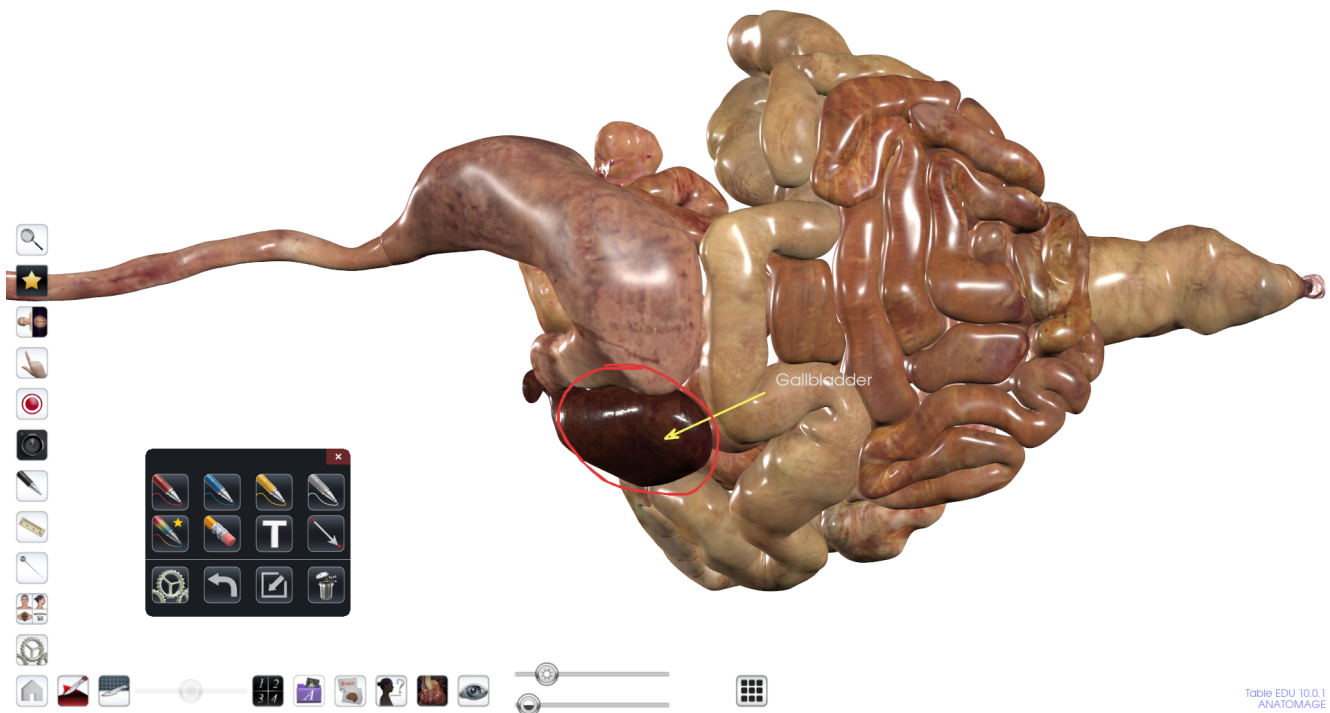
Step 6: Tap "Enter Text" to call a text dialog, then tap the white text bar to call the on-screen keyboard.



Step 7: Enter desired text and tap "OK".



Step 8: Tap the Arrow icon  then tap and drag toward desired region of Table to place arrow from tail to head.



Step 9: Tap the Erase Tool icon  and drag over desired region to erase.

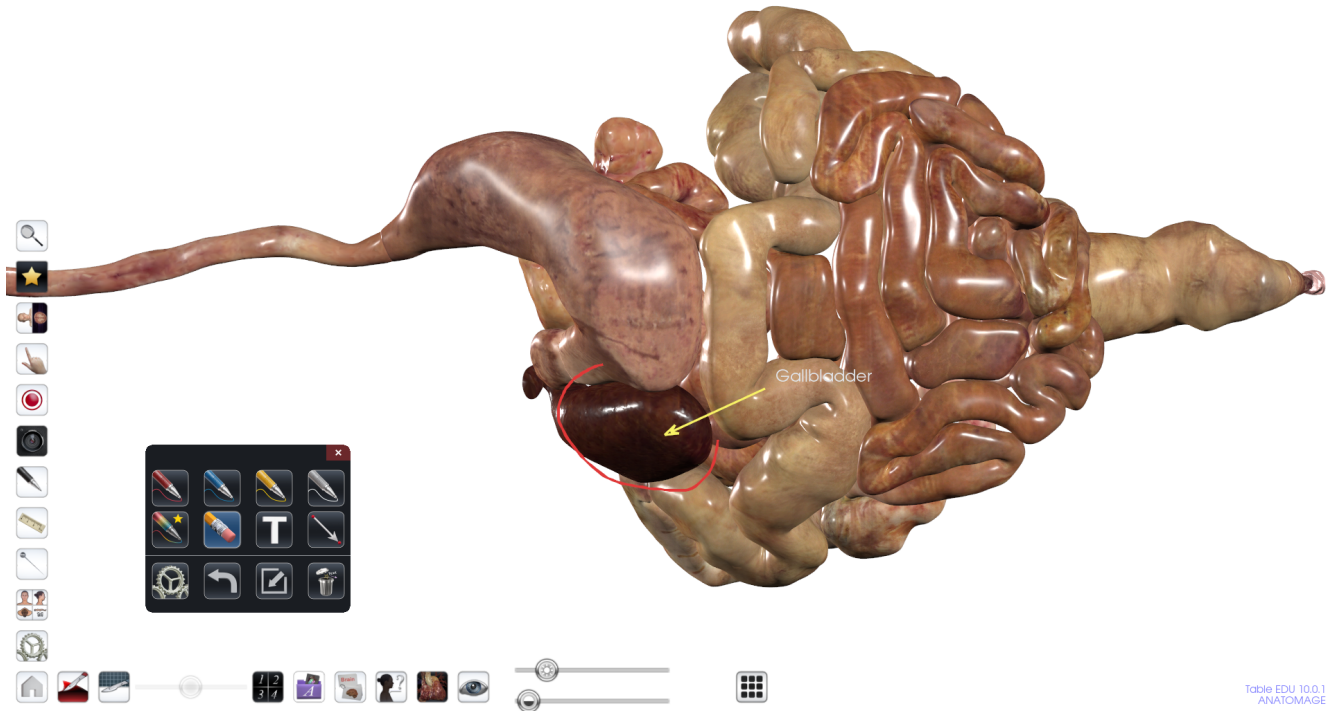





Table EDU 10.0.1  
ANATOMAGE

Step 10: Tap the Undo icon  to undo the last image you drew/placed. Tap the

Minimize icon  to minimize the pen menu. Tap the Delete icon  to delete all text and drawings.