

Anatomage

TABLE

Cadavers

How to Use Quiz Mode

Table of Contents

Create a Quiz	1
Edit a Quiz	7
Take a Quiz	8

Create a Quiz


Step 1: Select the Quiz icon



then select the Create Quiz icon



.

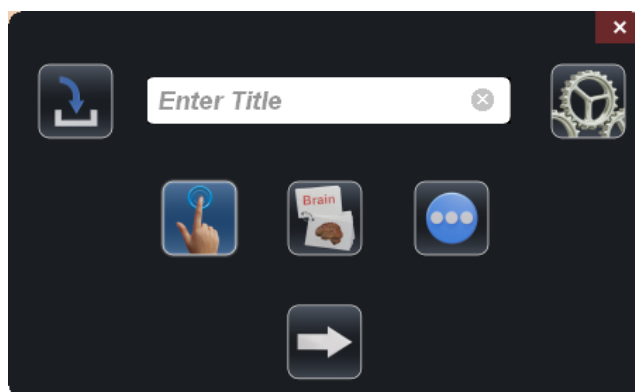

Step 2: Enter a title for the quiz.

Step 3: Select Settings to customize quiz settings. Check any of the options below to enable them. Uncheck them to disable.

Settings

Enabled Features

☒ Image Control

☐ Structure Slider Bars

☒ Cutting

General

☒ Repeat Skipped Questions

☒ Same Vessel Type Only

Highlight Color

☒ Randomize

☐ Protect with PIN

Student:

Current User

+

Timing

☐ Timer per Question

seconds

☐ Display Timer

☐ Total Quiz Time

minutes

☐ Display Total Quiz Time

Scoring

☐ Display Score

Points per Correct Answer

Negative Points per Incorrect

Points per Skipped

of Questions from Pool

Current Total Questions in Quiz = 1636

Apply

Some of these options include:

- Repeat skipped questions: If a student skips a question during the quiz, they can retake this question at the end.
- Same vessel type only: If you are quizzing on the nervous or cardiovascular system, allow for only the vessel type in question to be selected. For example, if the question is the femoral artery, students will only be allowed to click on arteries. Veins and nerves will not register.
- Randomize: The questions are randomly chosen each time the quiz is taken.
- Protect with PIN: Password protect any quiz file with a PIN. This PIN will be required to load in the file, edit the file, and start a quiz.
- Highlight color: Adjust what color structures on the cadaver will be highlighted during quiz questions.
- Student: Use the "+" icon to import in a student list in either.csv or .txt. When the students are taking the quiz, they will be able to choose their name from a drop down menu. This is optional, students will also be able to type their name in when taking a quiz, or leave this blank.
- Timers: set a timer per question or a total quiz time, and choose whether or not this time is displayed for the student.
- Display Results: At the end of the quiz, display the # correct, incorrect, and percentage for the quiz.
- Scoring: In the case of a game or an anatomy competition, for example, assign a point value to questions, and choose whether or not to display the score. Skipped questions can be positive and negative whole number values.
- # of Questions from Pool: Assign a certain number of questions from a total set. For example, your quiz file has a total of 50 questions, but you want each student to take only 20. Set this value to 20. Leave blank if you want all 50.
 - Note: The 20 questions will be randomly selected from the 50.

Once the settings are selected, tap "Apply" to return to the previous menu.

Step 4: Select the quiz type that you would like to create.



Pick Me will ask students to locate the indicated structure on the cadaver, and will reveal right and wrong results at the end.



Flashcards will ask students to locate the indicated structure on the cadaver, and will reveal the right and wrong results immediately after the answer is confirmed.



Tap the ellipse icon to expand and view other quiz options.



Highlight will light up the indicated structure on the cadaver. Students can test

themselves, then tap to reveal what the right answer is.



Multiple Choice will light up the indicated structure on the cadaver and prompt students with an automatically generated multiple choice list.




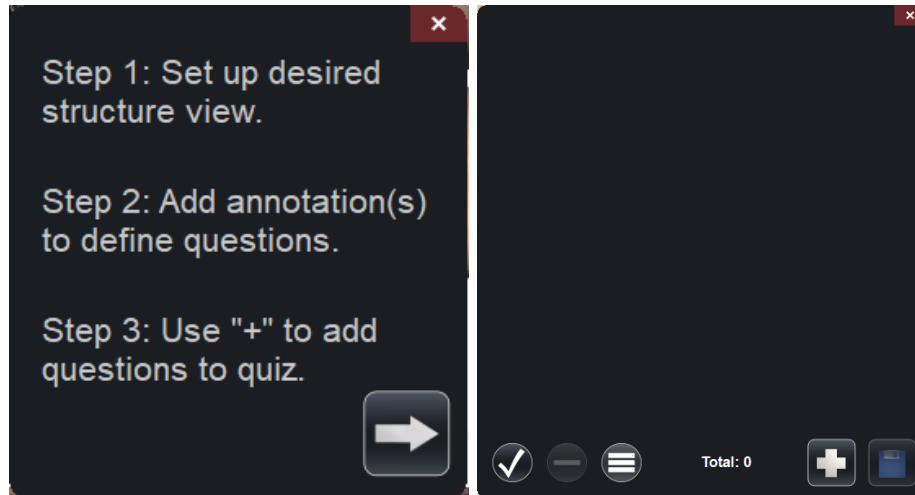
Game will convert the quiz into a game with 2, 4, 6, or 8 players. *(Tip: set a timer in the quiz settings to make games more high stakes!)*




Tournament Mode If you have ever participated in an Anatomage Tournament, this is the mode you can use to mimic those competitions. If you are interested in an Anatomage Tournament or to host one yourself, please contact us!


Step 5: Select the arrow icon  to continue to the next page.


Step 6: Read the instructions on the dialog. Then select the arrow icon  to continue to the next page.







Step 7: Use the Table as normal to set up your quiz view and questions. For example, use the

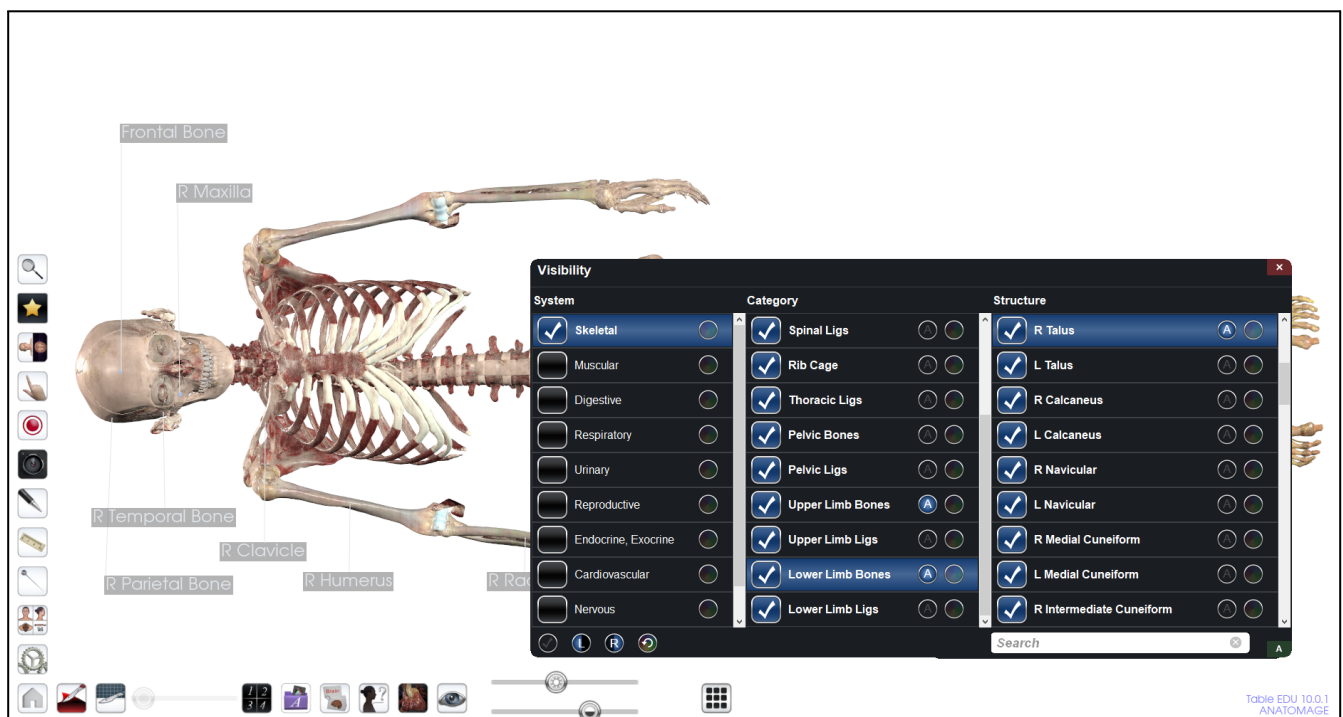Visibility menu  to set up the desired view and add annotations to define questions. (See “How to Use the Visibility Menu” document for more information.)

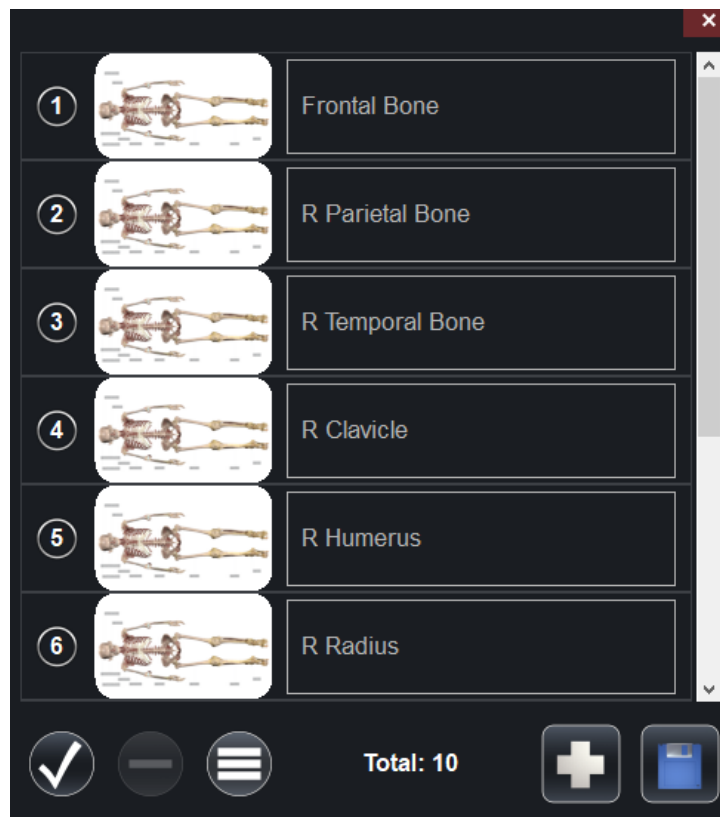

Step 8: Select the plus icon  to add the selected questions to the quiz.



Step 9: If you need to delete a question or reorder the questions, select the number key next to each image to select the question(s). Select the check icon  to select all questions. Select the minus icon  to delete the question(s). Engage the triple line icon  to adjust the order of questions.

Step 10: Select the Save icon  to expand all saving options. Tap the Quiz History icon



to save to quiz history, the Export icon



to save to the computer, or the Folder



icon to save to a folder within the Table software.


***Please note, you can also add bony landmarks and origin/insertions to a quiz by turning them on and annotating as you normally would.**

Creating Multiple Quizzes Back-to-Back in the same Session

Step 1: Follow the steps in **Create a Quiz** to create and save your first quiz.

Step 2: Select the Quiz icon  then select the Create Quiz icon .


This will automatically populate the quiz you just made in case you would like to edit that quiz.

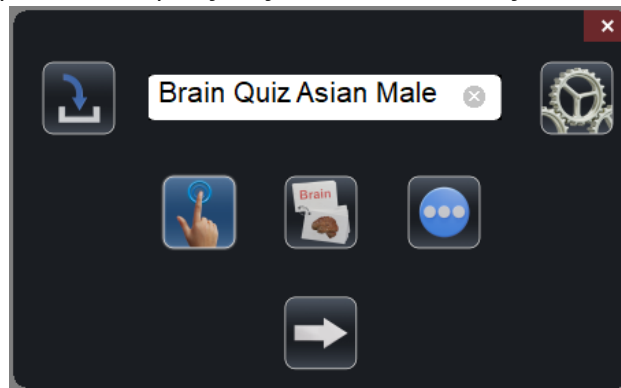





Step 3: To move forward with a new quiz, delete the title of the old quiz and input the new quiz name. This will not delete the old quiz, as you saved that in Step 1.

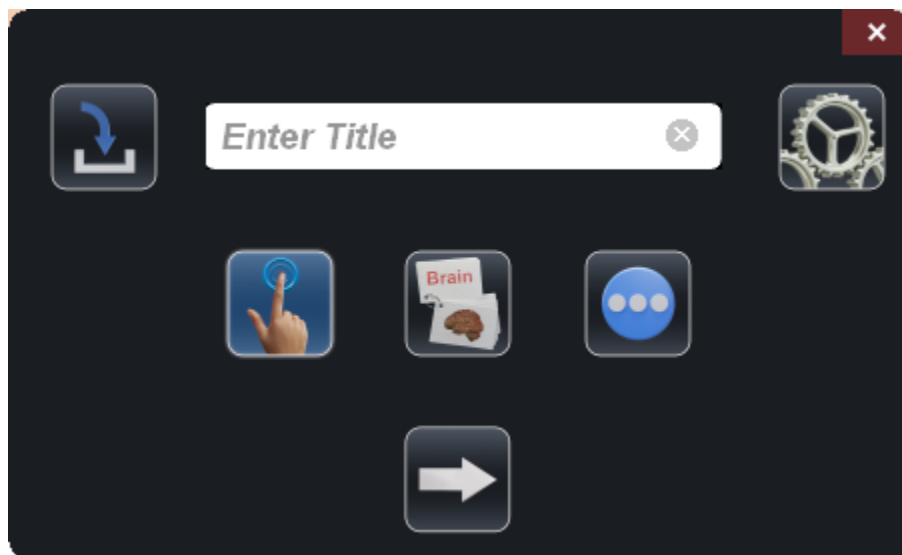Edit a Quiz

Step 1: Select the Quiz icon  then select the Create Quiz icon .





Step 2: Select the load icon  to expand all loading options. Tap the Quiz History icon  to load from quiz history, the Import icon  to import from the computer, or the Folder icon  to load from a folder within the Table software.

Step 3: Follow Steps 2-10 from “Create a Quiz” to edit and re-save the quiz.

Take a Quiz

Step 1: Select the Quiz icon  then select the Take a Quiz icon .

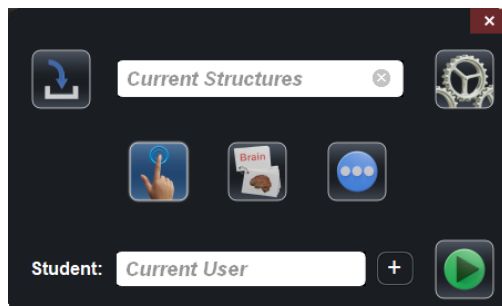


Step 2: Select the load icon  to expand all loading options. Tap the Quiz History icon



to load from quiz history, the Import icon











to import from the computer, or the

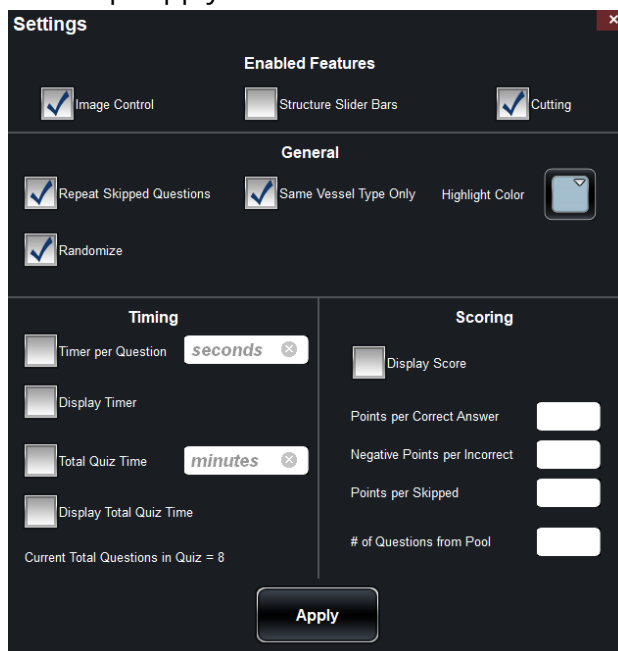Folder icon  to load from a folder within the Table software.

Alternatively, take a default quiz based on all structures currently in view on the rendering window without loading a previously saved quiz.

Step 3: Select Settings to customize quiz settings. Check any of the options below to enable them. Uncheck them to disable. Then tap "Apply".



Step 4: Select the quiz type that you would like to take. Select the Pick Me icon  or Flashcards icon . Tap the ellipse icon  to expand options and select the Highlight icon , Multiple Choice icon , Game icon  or Tournament Mode .

***Refer to page 2 and 3 for a description of each setting and quiz type.**

Step 5: Enter a student name or select the plus icon to load a student list text file.

Step 6: Tap the Start icon  to start the quiz.